




Empower™ ACTIVE Box Contents*

ACTIVE Sound Amplifier (left, right or pair)
Comply™ Premium Foam Tips† (for high volumes)
Package of Size 10 Batteries (yellow tab)
Cleaning Brush (with magnet & wire loop)
Carrying Case


*Exact quantities are determined by product ordered.
†Comply™ is a registered trademark of Hearing Components.

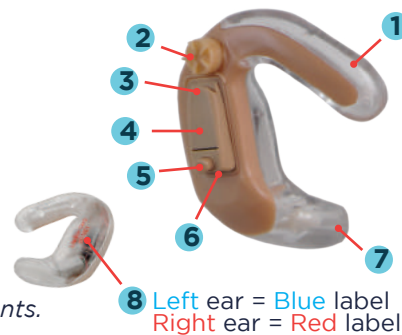

- 1 Arm
- 2 Volume Dial (on/off)
- 3 Nail Grip
- 4 Battery Door
- 5 Program Button
- 6 Microphone
- 7 Receiver
- 8 Label

Step 1: Insert battery

- a. Remove yellow tab from a size 10 zinc-air battery. To increase battery life, wait one minute to allow battery to fully charge before inserting battery.
- b. Gently open battery door (do NOT pass 90°). Locate red + sign on battery door.
- c. Insert battery on an angle, positive + side up and tilt back into place.
- d. Carefully close battery door. If there is resistance closing door, the negative side of battery may be up.

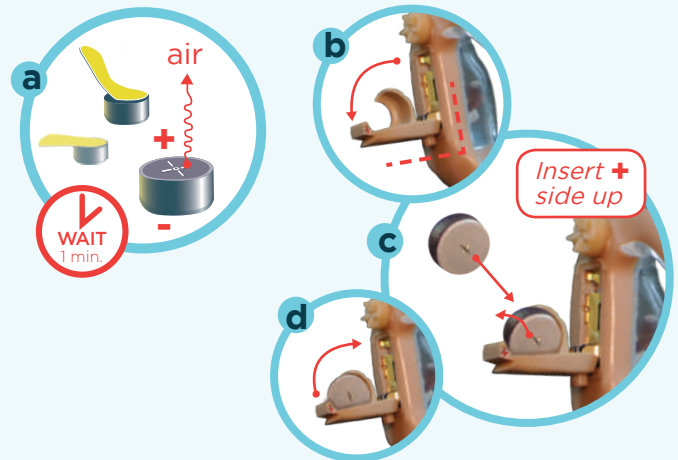


Step 2: Place ACTIVE in ear

Hint: Blue label on backside indicates left, red label = right.

- a. Gently place receiver in entrance of ear canal.
- b. Carefully tuck arm into upper bowl of ear.

Please note: The ACTIVE (without a tip) allows sounds from the environment to join the amplified sounds for a more natural listening experience.

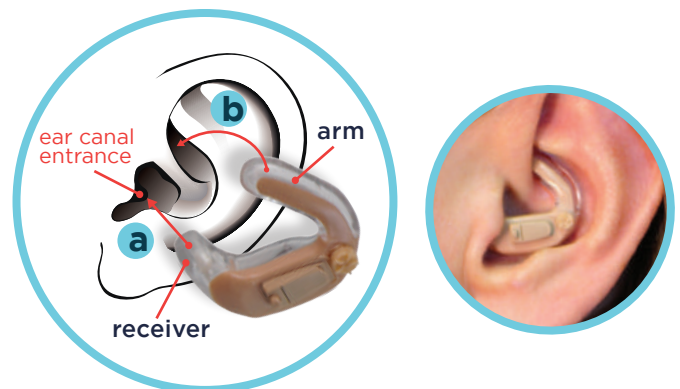

Using Optional Comply Premium Foam Tip

Use foam tips to prevent whistling at higher volumes and/or to provide a more snug fit.

- a. Push tip completely onto receiver until you can feel the end of receiver through the opening in the tip.
- b. Pinch tip between finger and thumb to compress foam.
- c. Let compressed tip expand in ear canal entrance and tuck arm into upper bowl of ear (see Step 2 above).

